

Gebrauchsanweisung | User Manual

Transatflow Multi-APS safety

Sicherheits-Füllsystem in Verbindung mit Patientenleitung mit integrierter Keimsperr Transatflow PWL und PWLS

Safety Filling System for patient replacement tube with integrated anti-bacterial barriers Transatflow PWL and PWLS



1. Product Overview



Multi-APS safety

REF
2590313
2590323



Patient replacement tube Spiraled PWL with integrated anti-bacterial barriers



Patient replacement tube Standard PWL with integrated anti-bacterial barriers

REF
2580424
2580436

2. Instructions



1.

Connect the yellow tube to the contrast-agent bag/ container.



2.

Connect the blue tube to the NaCl bag/ container.




3.

Secure the Multi-APS Safety system onto the syringe plungers of the injector:


Yellow = Contrast Agent
Blue = NaCl

Fill the syringes by means of the injector.
(Follow the manufacturer's instructions for the injector and syringes).

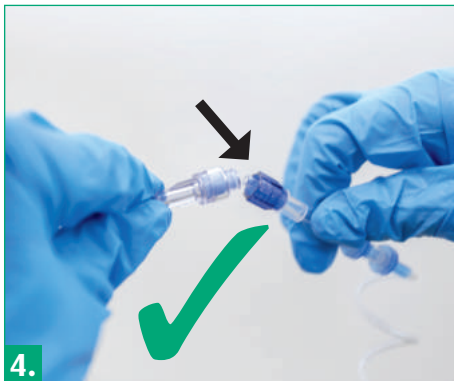

4.

Connect the blue end of the patient replacement tube with the blue safety valve of the Multi-APS system and begin to bleed the fluid through.

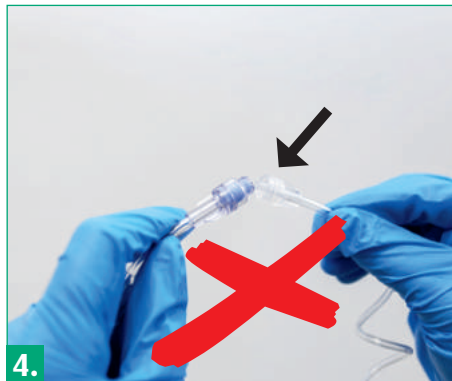

4.



5.

Securely connect the patient replacement tube to the patient's intravenous cannula.

IMPORTANT!

- » Only apply pressure to the system when the patient replacement tube is closed
- » Ensure that no pressure is being applied to the APS system before disconnecting it from the patient replacement tube.
- » The patient replacement tube must be disposed of after every use.
- » Before connecting a new patient replacement tube to the device, the safety valve must be properly disinfected and dried.
(We recommend the Disinfecting Agent from Medical Products)
- » If the closed system becomes disconnected from the IV fluid bag or syringe, a new device must be used.
- » The device is suitable for one-time use in the administration of contrast agent or NaCl across a period of 8 hour.
After 8 hours the system must be removed.
- » The system must not be re-used or recycled.

Vertrieb durch / Distributed by:

Transatlantic Handelsgesellschaft Stolpe & Co. mbH
Siemensstraße 21-23
61267 Neu-Anspach, Germany
Tel.: +49 (0)6081 94 30 50
Fax: +49 (0)6081 94 30 80
info@transat.de
www.transatlantic.de

🏭 Hersteller / Manufacturer:

CARDEA GmbH & Co. KG
Bruchweg 23
38822 Emersleben, Germany